SECURITY

Modelos: - VIC821/VIC821V FFP1 NR D
- VIC821/VIC821V FFP2 NR D

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EN PARTICULE RESPIRATOR

- Disposable particulate respirator
- Filters out bacteria and viruses
- Remove the suspended particulate
- Dust, pollen protection
- High filter efficiency
- Tight sealed
- Comfortable use
- Latex-free head strap

IMPORTANT

Particulate respirators are suitable for providing effective protection against solid and liquid aerosols. There are three categories according to leakage and maximum penetration through the filter material: FFP1, FFP2 and FFP3. The user is responsible for selecting the appropriate respirator to provide protection against the type and concentration of the contaminant(s) in the working area.

Use instructions

1. Failure to follow all instructions and limitations can seriously reduce the effectiveness of the respirator and could lead to illness, injury or death.
2. A properly selected respirator is essential. Before occupational use, the wearer must be trained by the employer in the correct use of the respirator in accordance with applicable safety and health standards.
3. This product does not supply oxygen. Use only in adequate ventilated areas containing sufficient oxygen to support life.
4. Discard the respirator and replace with a new one if:
 - a. Excessive clogging of the respirator causes breathing difficulty;
 - b. The respirator becomes damaged;
 - c. The user experiences symptoms such as dizziness, irritation or other distresses occurs.
5. Leave the contaminated area if dizziness, irritation or other distresses occurs.
6. Keep the respirators in the display box away from direct sunlight until use.
7. Users must be fit tested prior to wearing this respirator either qualitatively or quantitatively according to 29CFR 1910.134.
8. Before wearing the respirator users must check whether there is crack or spot around the welding spot. If yes, the mask must be discarded at once and cannot be used again.
9. Keep the unused respirators in their closed boxes and store in a dry non-contaminated area between +5°C and +38°C. Storage life is 3 years from date of manufacture.
10. *NR* indicate not reusable (single shift only).

Use limitations

1. Do NOT use the respirator or enter or stay in a contaminated area under the following circumstances:
 - a. Atmosphere contains less than 19,5% oxygen;
 - b. If you smell or taste contaminants;
 - c. For protection against gases or vapour;
 - d. Contaminants or their concentration are unknown or immediately dangerous to life or health;
- e. Concentrations of contaminants exceed maximum use concentrations in applicable government regulations or 12 times the PEL (Permissible Exposure Limit) whichever is lower;
- f. For sandblasting, paint-spray operations and asbestos;
- g. Use in explosive atmospheres.
2. Do NOT modify or misuse the respirator.
3. Do NOT use the respirator with facial hair or any other conditions that may prevent a good face-seal.
4. There has to be an advice that this mask can only be used for one work shift.

Fitting instructions

1. Place fingers inside the respirator. Bend the nose clip around fingers to form nose shape.
2. Hold the respirator in position over the nose and mouth with the head straps around the back of the hand. Pull lower strap over the head and fit around the back of the neck.
3. Pull the upper strap over the head positioning it just above the ears.
4. Using both hands, for the nose clip around the nose and shift the head straps to ensure a positive face seal.
5. Seal check:
 - o To test the fit of a respirator without an exhalation valve, cup both hands over the respirator and exhale sharply.
 - o To test the fit of a respirator with an exhalation valve, cup both hands over the mask and inhale sharply. A negative pressure should be detected inside the respirator.
 - o If air flow is felt in the nose area, re-adjust/tighten the nose clip.
 - o If air flow is felt around the edges of the respirator, re-position the respirator/head strap to achieve a better fit.
6. Change the mask immediately if breathing becomes difficult or mask becomes damaged or distorted.
7. Change the respirator if a proper face seal cannot be achieved.
8. Careful observance of these instructions is an important step in safe respirator use

See information supplied by the manufacturer

Temperature range of storage conditions

Maximum relative humidity of storage conditions

WARNING:

This respirator helps protect against certain particulate contaminants but does NOT eliminate exposure to or the risk of contracting disease or infection. Misuse may result in sickness or death.

Manufacturer:

CENTRAL LOBÃO – Ferramentas Eléctricas S.A.
Rua da Gandara no. 664, 4520-606 S. João de Ver, S. M. Feira, Portugal

This filtering respiratory are in conformance with the provision of PPE Regulation 2016/425 and, where such is the case, with the harmonized standard EN149:2001 is identical to the PPE which is the subject of EC type examination certificate no.: IFA 1801322 / 1801323 / 1801324 / 1801325 as issued by:
INSTITUT FÜR ARBEITSSCHUTZ DER DEUTSCHEN GEZETZLICHE UNFALLVERSICHERUNG, Alte Heerstr 111, 53757 Sankt Augustin Deutschland (notified body: O121) as a Category III PPE, the products are subject to the procedures as set out module C2 as PPE regulation.

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